



# DISINTEK™ OPA TEST STRIPS

For CIDEX® OPA or MetriCide® OPA Plus™ High-level Disinfectants

**Product Code:** 5121

**Description:** DISINTEK OPA TEST STRIPS

**Packaging:** 2 bottles/kit; 50 strips/bottle

## Directions for CIDEX® OPA:

1. Immerse entire indicator pad of test strip into solution of **CIDEX OPA** for a full twenty (20) seconds.
2. Remove strip and touch the long edge on a paper towel to remove excess OPA solution.
3. Lay strip, pad side up, on a paper towel.
4. **Wait an additional forty (40) seconds** [after the twenty (20) second immersion in CIDEX OPA], then interpret results by comparing the color of the indicator pad to the color block.

## Directions for MetriCide® OPA PLUS™:

1. Immerse entire indicator pad of test strip into solution of **MetriCide OPA PLUS** for a full five (5) seconds.
2. Remove strip and touch the long edge on a paper towel to remove excess OPA solution.
3. Lay strip, pad side up, on a paper towel.
4. **Wait an additional fifty-five (55) seconds** [after the five (5) second immersion in MetriCide OPA Plus], then interpret results by comparing the color of the indicator pad to the color block


## INTERPRETATION OF RESULTS:

**PASS** – the indicator pad is similar to or greener than the PASS color block

**FAIL** – the indicator pad is blue

CIDEX OPA is a registered trademark of Advanced Sterilization Products, a Johnson & Johnson company.

MetriCide OPA Plus is a registered trademark of Metrex Research Corporation, an indirect subsidiary of Sybron Dental Specialties, Inc.

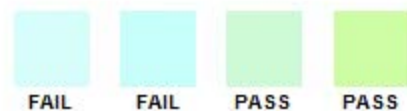

Diagrams are for illustration purposes only.  
Always use the color blocks on the actual bottle label to interpret results

## Your Independent Choice for HLD Test Strips



Toll Free: 1.800.542.4670  
Phone: 1.574.264.3440  
Fax: 1.574.266.6222  
Website: [www.serim.com](http://www.serim.com)  
E-mail: [customerservice@serim.com](mailto:customerservice@serim.com)